



### Daily Maintenance

#### Start of day

- Flush water lines &/or add cleaning tablets
- Clean dental chair, unit & stool
- Check sterilizer water level, add distilled water only if needed

#### Between patients

- Clean, lubricate & sterilize handpieces and prophy/contra-angles, or replace disposables

#### End of day

- Remove & replace saliva ejector screen
- Remove/replace vacuum solids trap on unit arm
- Sanitize cuspidor bowl & replace screen
- Flush vacuum lines
- Perform computer backup procedures & verify completion
- Drain used ultrasonic solutions & refill
- Turn "OFF" all electrical switches for air compressor, vacuum pump(s), master water control valve, medical gas system and any other manual valves.

### Weekly Maintenance

- Clean handpiece fiber-optics
- Clean dental stool caster wheels
- Clean dental light reflector & lens shield (ensure light is cool)
- Clean sterilizer chamber, drain & refill water reservoir
- Clean particle filter inside autoclave chamber
- Perform sterilizer spore testing
- Clean & oil lab handpieces & engine
- Remove/replace solids filter at vacuum pump (water-type pumps)
- Check compressor oil level (oil-type compressors; add if needed or call for service)
- Check compressor dryness indicator color (blue is good; if pink, call for service)
- Check compressor for air leaks (if found, call for service)

### Monthly Maintenance

- Test dental unit water CFU level & treat waterlines as needed
- Lubricate HVE, saliva ejector & water bottle o-rings
- Replace & clean out plaster trap
- Perform monthly cleaning of sterilizer
- Run ultrasonic foil performance evaluation test
- Check compressor and vacuum pumps for leaks (if found, call for service)
- Check amalgam separator fill level (if full, replace cartridge or call for service)

### Annual Maintenance – call for annual maintenance servicing

- Replace unit air & water filters in the floor box or utility area
- Replace model trimmer wheel
- Replace amalgam separator cartridge
- Replace compressor intake filters &/or dryer filters
- Drain & replace compressor oil (oil-type compressors)
- Replace vacuum system oil & filters (if oil is used)
- Replace master water control valve filter element/cartridge